## Obs!

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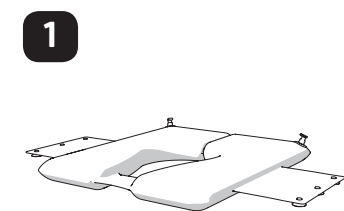
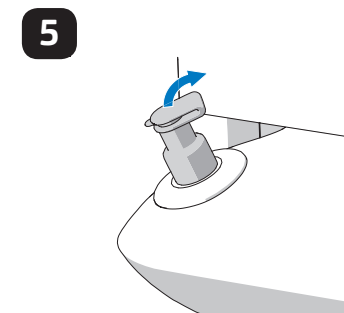
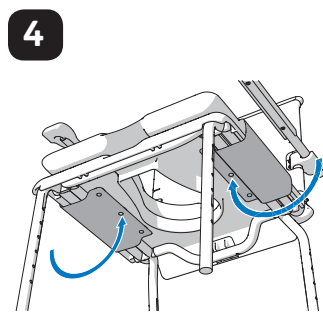
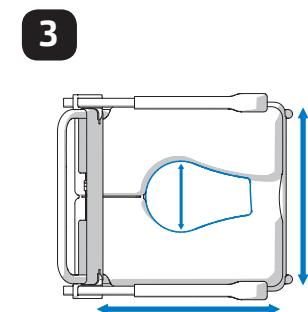
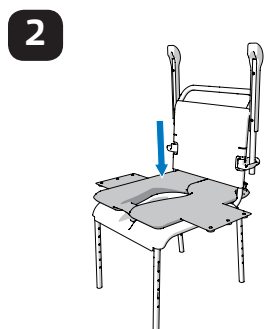
## EN User manual Vicair® Liberty Hygiene

### 1. Installation

1. Place the cushion on the toilet chair with the valves face up at the rear end. (2)
2. Check if the cushion fits the dimensions of your body and toilet chair and make sure the opening in the cushion is direct above the opening in your toilet seat. (3)
3. Pull the fixing flaps around the sides of the seat of your toilet chair. Press the suction cups firmly against the bottom of your toilet chair. (4)
4. Open the valves (5) on the rear of the cushion and sit on the cushion for 5 minutes (6) to allow the cushion to adapt to your body shape and weight.
5. Close the valves. The cushion is ready for use.

### 2. Use

**Warning!** Do not use sharp objects and do not use or leave the cushion close to a naked flame or high temperature heat source.

- When the cushion feels unstable during use, repeat actions 4 and 5 as described in paragraph "Installation".
- When transferring the cushion to another user disinfect the cushion.

### 3. Cleaning

**Warning!** Do not wash, steam clean or autoclave the cushion.

**Warning!** Do not use aggressive or abrasive cleaners to clean the cushion.

- Light stains can be wiped off using a damp cloth and mild detergent.
- For disinfecting the cushion use an anti-bacterial agent.

### 4. Warranty

Vicair offers a limited warranty against defects in workmanship and materials for a period of two (2) years after the original date of purchase. For general warranty conditions see our website

[www.vicair.com](http://www.vicair.com)

8 717261 235028